

Durable, Lightweight and Portable




Hi-Torque front suspension

The i-Go from Pride Mobility features an advanced folding technology, enabling it to be transported with ease. The i-Go features front suspension, a durable seating system, under seat storage and much more. Designed to fit inside any small space, the i-Go is the perfect choice for the active individual.



Specifications

| | |
|---|---|
| WEIGHT CAPACITY | 120 kg |
| MAXIMUM SPEED ² | Up to 6 km/h |
| GROUND CLEARANCE ¹ | 44 mm |
| OVERALL LENGTH ¹ | 876 mm |
| OVERALL WIDTH ¹ | 600 mm |
| TURNING RADIUS ¹ | 900 mm |
| STANDARD CONTROLS | Micon Control System |
| TOTAL WEIGHT (WITHOUT BATTERIES) | 19,8 kg |
| BATTERY WEIGHT (EACH) ¹ | 1,35 kg |
| SUSPENSION | Front |
| BATTERY REQUIREMENTS | 2 x 6.6 Ah, Li-ION Battery |
| SEATING | Saddle back seat |
| TIRES | Front castors: 7" solid Rear castors: 8,5" solid |
| COLOUR |  Silver |

Notes:

- 1) Due to manufacturing tolerances and continual product improvement, this specification can be subject to a variance of + or - 3 %.
- 2) Specification varies with user weight, terrain, battery type, battery condition and tire pressure.
- 3) Battery weight may vary based on manufacturer.

Note: All specifications subject to change without notice.

The information contained herein is correct at the time of publication; we reserve the right to alter specifications without prior notice. Speed varies with user weight, terrain type, battery charge, battery condition and tire pressure.

Features

- Large secure storage compartment
- Unique comfort seating
- HI-Torque front suspension
- Micon Control System
- Under seat storage

How to fold



- 1 Switch off and remove joystick



- 2 Place joystick in backrest pocket and remove cushion



- 3 Fold backrest down, then pull both arm rests anticlockwise toward the back



- 4 Pull the two black toggles (located on the back) together



- 5 Push down on the seat frame until unit fully flattens



- 6 The i-Go is now ready to go!